

POWERLIFT 1000MOB



POWERLIFT 1000 MOB

The POWERLIFT 1000MOB BARIATRIC LIFT AND TRANSFER-ARM STYLE by Wheelchairs of Kansas is the traditional style, portable lift constructed of heavy-duty steel.

The easy rolling casters allows to position the lift into varies locations with all six that swivel. The lift is operated by a four-button hand control and a linear driven actuator that is practically silent. For added safety the emergency release allows the individual to be lowered when battery powered is interrupted.

By the simple touch of a button, the base width adjusts automatically for stability. This lift is battery powered with a internal charger. The weight capacity can be 750 lbs. or 1,000 lbs. by simply adjusting the slider clamp.



BATTERY

To operate lift, turn battery on by pressing the red button. Press the red button again to turn the battery off. This lift runs off a 24V Gel battery. To charge the battery, plug the lift into a 115V outlet with the cord.



WEIGHT ADJUSTMENT SLIDER

To change the weight capacity, loosen the black knob. Move slider clamp to desired weight capacity. Tighten knob when the slider clamp is in place.



EMERGENCY LOWER RELEASE

The lift has a emergency lower release in the event of electrical failure. Locate the red clip on the base of the actuator rod, slide up to activate the emergency release.